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DE

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#### Ingredients:

Sodium hyaluronate 1 mg/ml, sodium chloride, sodium hydrogen phosphate, chlorides (potassium, calcium, magnesium), sodium hydrogen carbonate, purified water

#### Areas of application:

For lubrication of the eye surface in case of dry, irritated or burning eyes and for the relief and treatment of irritations due external factors such as dry or air-conditioned environment, smoke, dust, pollen as well as after surgery on the eye.

The eye drops can also be used with hard or soft contact lenses. Can be used with contact lenses.

#### Contraindications:

The eye drops should not be used in the presence of a hypersensitivity to any of the ingredients.

#### Notes:

- Please note that your vision may be temporarily affected immediately following the use of the product
- Do not use the eye drops if the bottle has been damaged
- Do not use the eye drops simultaneously with other eye medicines use because the effect of the preparations may be altered. In principle, if other eye drops/ointments are used, an interval of approx. 30 minutes between using the different products should be adhered to.
- In order to avoid contamination of the external surface, please ensure that you avoid touching the surface of the eye or other items with the tip of the dropper.
- In very rare cases, calcification of the cornea in association with the use of eye drops with a high phosphate concentration has been reported in patients with significant damage to the cornea. The eye drops contain only a physiological concentration of phosphate and so their application, in principle, is not restricted to existing severe corneal disorders/damage only. If other eye drop preparations are also used, however, their phosphate concentration should be borne in mind in this regard since, particularly in the case of anti-glaucoma agents or anti-allergy agents, they may contain high phosphate concentrations.

#### Tips for use:

- Drop the eye drops onto the surface of the eye or onto the conjunctival sac.
- Washing the hands thoroughly before use is recommended
- Remove the tamper-evident seal before first use (see Fig. 1)
- Remove the screw cap before each use (see Fig. 2)
- Tilt your head back a little (see Fig. 3)
- Hold the bottle vertically over each eye in turn, with the opening facing downwards (see Fig. 3)
- Carefully open your eye using the thumb and index finger of your free hand or gently pull the lower lid down (see Fig. 3)
- By gently squeezing on the bottle, 1 drop is instilled onto the surface of the eye or onto the conjunctival sac (see Fig. 4)
- Repeat the application for the other eye
- Put the protective cap carefully back on immediately after each use (see Fig. 5)

#### Directions for use:

The eye drops can be used as required. In general, application up to 4 times per day is recommended.






#### Side effects:

Hypersensitivity reactions may occur in very rare cases. In this case, the product should not be used again.

#### Shelf-life and storage:

Store in a cool and dry place.  
Keep out of reach of children.  
Can be used for 6 month after first use.  
Do not use after the expiry date.

Date of last revision: 2018-05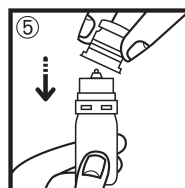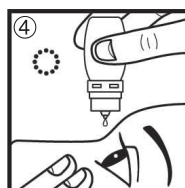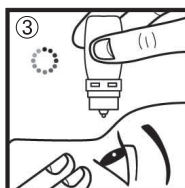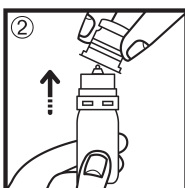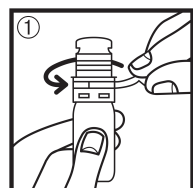

Symbol/Symbol/Simbolo/Symbol:

|  |   |   |  |  |
|--|---|---|--|--|
|  | Hersteller                                      | Fabriqué par                                | Prodotto da  | Manufacturer                             |
|  | Verfalldatum<br>(JJJJ-MM: Jahr-Monat)           | Date de péremption<br>(AAAA-MM: Année-Mois) | Data di scadenza<br>(AAAA-MM: Anno-Mese)               | Expiry date<br>(YYYY-MM: Year-Month)     |
|  | LOT-Nummer                                      | Numéro de lot                               | Lotto numero   | LOT-number                               |
|  | Lagertemperatur<br>2°C – 25°C                   | Température de stockage<br>2°C – 25°C       | Temperatura di conservazione<br>2°C – 25°C             | Storage temperature<br>2°C – 25°C        |
|  | Gebrauchsinformation beachten                   | Observer la notice explicative              | Legerge attentamente il foglio illustrativo            | Observe instructio leaflet               |
|  | Nicht verwenden wenn die Flasche beschädigt ist | Ne pas utiliser si le flacon est endommagé  | Non usare se la bottiglia è danneggiato                | Do not use if the bottle is damaged      |
|  | Nach Anbruch 6 Monate verwendbar.               | A utiliser dans les 6 mois après ouverture. | Può essere usato fino a 6 mesi dopo il primo utilizzo. | Can be used for 6 month after first use. |